



Element DCX™ Slide Transfer and ISE Cleaning

Slide Transfer and ISE Cleaning


1. Press the Gear Icon, choose Normal Mode

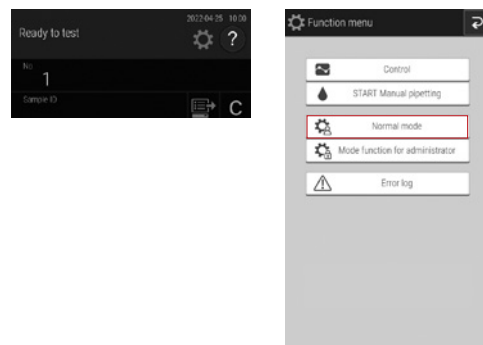



2. Select Cleaning mode, Start Cleaning and when prompted to power down, do not power down. Leave the machine on

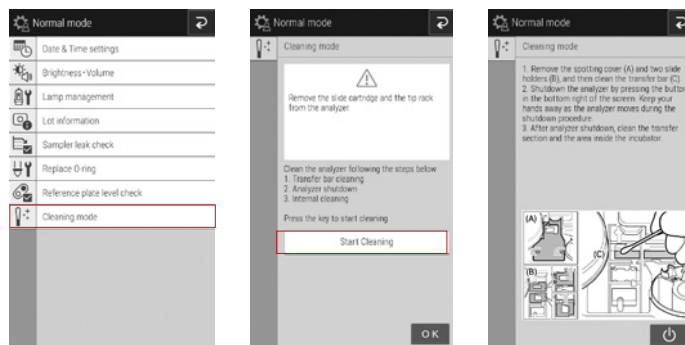


3. Remove the Transfer Cover by loosening the two thumb screws as shown.

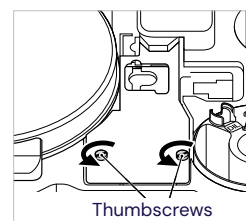

4. Remove the slide weights, clean with alcohol and a wipe to remove any plasma and debris from both sides.

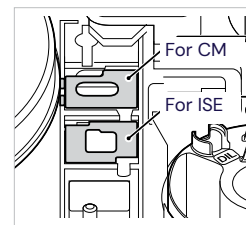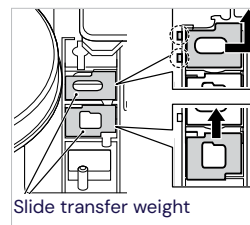

5. Clean the spotting area and transfer bar with distilled water and a cotton swab or wipe.

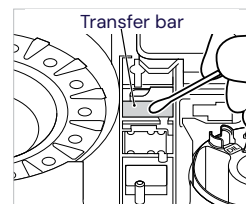

6. Clean the ISE pins with dry cotton swab. Use a flashlight to inspect the brass electrodes and gently remove any dust or debris that is trapped in this area. Also ensure the plastic surface is free from plasma and dirt.
7. Return parts in reverse order to reassemble after cleaning. Make sure the slide weights are returned to their proper positions as shown below.
8. Press the back button to exit cleaning mode.

